

You Dont Have To Be A Trained Paramedic To **Save A Life**



iPAD

The Intelligent Public Access Defibrillator

A British Heart Foundation nominated device

Issue Three



Setting Standards Across Seven Sectors

iPAD SP1

Overview

The iPAD – SP1 will redefine your expectations for a defibrillator. It has many new innovative features including pre-connected smart electrodes, “i” button technology, ambient noise detection, but most of all, its designed to be used by minimally trained individuals.

1) Patient Mode Switch

- Easily switch from adult to child mode without changing pads
- Safety cover prevents accidental switching.

2) Speaker

- Ambient noise detector measures level of background noise and adjusts the volume of the voice prompts accordingly.
- Ideal for noisy environments such as public places, factories, warehouses, schools etc.

3) Pad Connector

4) Power Button

5) Shock Button

6) Smart Pads Storage

- Electrode pads are stored, pre-connected, in a clear compartment on the underside of the unit.

7) Disposable Battery Pack

- High Capacity
- Type DC 12 Volt 4.2Ah, Lithium Manganese Dioxide Capacity
- Minimum 200 shocks (150J)
- Lifespan 5 years assumes unit is kept within operable temperature range and after the initial check, the unit is left in standby mode.

8) Data Recording

- IrDA Port - Wireless Transmission of data to PC
- SD Card Port - Data can be transferred easily
- Internal Memory (Nand-Flash) - ECG, Event
- Storage Capacity - Multi recording
- 5 events / max 3 hours
- Data review
- program CU-EX1
- Software (for PC)



The iPAD SP1 defibrillator from CU Medical Systems is designed for use by people with minimal training in the event of a cardiac arrest. Easy to use with built in adult/child switch.

9) Visual Indicator

- Visual Indicator shows at a glance
- Battery Life
- Unit Status
- Pad Status

10) Information Button

'i' button performs the following functions;

- Allows responder to select if CPR metronome is heard during CPR
- Checks for faults and errors
- Provides 'handover' information to Ambulance crew (usage time and shocks delivered)
- Allows the SP1 to be programmed with CPR protocols such as compression rate, number of compressions, breaths and cycles etc, ensuring that the SP1 is always up to date
- Downloads data for review
- Checks the units software version

11) Indicator Lights

- Indicator Lights operate in time with the voice prompts to prompt the responder with;
 - Pad placement
 - Staying clear of casualty
 - CPR compressions



AED Comparison Table

	IPAD SP1	Cardiac Science G3 Pro	Philip/Laerdal FR3 Pro	ZOLL AED Plus	Physio Control Lifepak CR+	HeartSine Samaritan
Electrode Tested For Presence	✓	✓	✓	✗	✗	✗
Daily, Weekly, Monthly Self Tests	✓	✓	✓	✓	✓	✓
All Tests Automated	✓	✓	✓	✓	✓	✓
Status Indicator Shows Clinical Readlines	✓	✓	✓	✓	✓	✓
Battery Capacity Gauge	✓	✓	✗	✗	✗	✗
Pad Life Indicator	✓	✗	✗	✗	✗	✗
Used Pad Detection	✓	✓	✗	✗	✗	✗
Pad Life From Manufacture	30 months	2 Year	2 Years	5 Years	2 Years	3.5 Years
Uses Same Sets of Pads for Child and Adult	✓	✗	✓	✗	✗	✗
Ambient Noise Detector	✓	✗	✗	✗	✗	✗
Intergarted Pad Storage	✓	✗	✗	✓	✓	TBC
CPR Detection	✓	✗	✗	✓	✗	✗
Illiminated Prompts	✓	✗	✓	✓	✓	✓
Internal Mermory Capacity	15 Hours	1 Hour	TBC	7 Hours	20 Minutes	45 Minutes
Data Transfer Method	IR / SD Card	Direct Connect	Direct Connect	Direct Connect	Direct Connect	Direct Connect
User Reprogrammable	✓	✓	✓	✓	✓	✓
CPR Metronome	✓	✓	✓	✓	✓	✓
Paramedic Handover Information	✓	✗	✗	✗	✗	✗
Fluid Ingress Protection	IP 55	IP 24	IP 55	IP55	IP 55	IP 56
Battery Capacity (Shocks/Shelf)	2000/5 years	290/4 years	/5 years	/5 years	/2 years	/3.5 years
Carry Case Supplied As Standard	✓	✗	✗	✓	✓	✓
Manual Self Test Option	✓	✗	✗	✗	✗	✗
All Status Indicators Visable When Case is Closed	✓	✗	✗	✗	✗	✗

Ordering Details

Code	Description
1844	iPAD - CU SP1 GENUINE OEM

Contents

2 x AED Starter kits
1 x AED Carry Case



Code	Description
1850	Alarmed Aed Cabinet
1851	Wall Mounting Bracket
1855	Replacement Battery
1845	Adult/Child Dual Use 'Smart' Pads
1846	Child Pads
1849	Trainer Unit
1847	Training Electrodes (Adult)

1850 Alarmed
AED Cabinet



1851 Wall Mounting Bracket




1845 Adult/Child
Dual Use 'Smart' Pads



1855 Replacement Battery



Call us on **0161 902 3030**, email enquiries@steroplast.co.uk or visit www.steroplast.co.uk for more information.

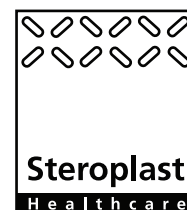

Steroplast Healthcare Limited, Alpha Point, Bradnor Road, Manchester, M22 4TE, England
Tel: 0161 902 3030 Fax: 0161 902 3040 Email: enquiries@steroplast.co.uk Web: www.steroplast.co.uk

Setting Standards Across Seven Sectors

Emergency

Hospital

Sport

Veterinary

Medical

First Aid

Retail